## Description

Tetric® N-Bond Universal is a light-curing single-component dental adhesive for enamel and dentin, which is supplied in the VivaPen® and bottle for repeated dispensing of the adhesive. The adhesive can be used with the:

- Self-etch technique
- Selective-enamel-etch technique
- Etch & rinse technique

## Composition

Tetric N-Bond Universal contains methacrylates, ethanol, water, highly dispersed silicon dioxide, initiators and stabilizers.

## Indication

- Direct-placed light-curing composite and compomer restorations.
- Direct-placed core build-ups with light-, self- and dual-curing composites.
- Repair of fractured composite and compomer restorations.
- Adhesive cementation of indirect restorations with light- and dual-curing luting composites.
- Sealing of prepared tooth surfaces before temporary / permanent cementation of indirect restorations.
- Desensitization of hypersensitive cervical areas.

## Contraindication

- Do not use Tetric N-Bond Universal if the patient is known to be allergic to any of the materials' ingredients or if the stipulated working technique cannot be employed.
- Applications in which sufficient illumination cannot be ensured (e.g. cementation of root canal posts).
- Direct pulp cappings.

## Side effects

In rare cases, components of Tetric N-Bond Universal may lead to sensitization. In such cases, Tetric N-Bond Universal should no longer be used.

## Interactions

- Materials containing peroxides (e.g. bleaching agents or oxidizing disinfectants) may inhibit the polymerization of adhesives. Consequently, the application of such materials together with Tetric N-Bond Universal must be avoided.
- Phenolic substances (e.g. eugenol) inhibit polymerization of Tetric N-Bond Universal.
- Iron- and aluminium-based styptic agents must not be used shortly before or during the application of Tetric N-Bond Universal.
- Fluoridation agents must not be used prior to the application of Tetric N-Bond Universal.

## Procedure for direct and indirect restorations

### 1. Isolation

A dry operating field must be ensured. Ideally, a rubber dam should be placed (e.g. OptraDam® Plus).

### 2. Pulp protection / cavity liner

In very deep cavities, areas close to the pulp must be selectively coated with a calcium hydroxide liner (e.g. ApexCal®); and subsequently covered with pressure-resistant cement (e.g. glass ionomer cement).

### 3. Conditioning with phosphoric acid gel (optional)

The bond to enamel can be improved by selectively etching the enamel or by applying the "etch & rinse" technique. Unprepared enamel surfaces must be conditioned with phosphoric acid (e.g. N-Etch). Please observe the Instructions for Use for the phosphoric acid gel.

#### (a.) Selective-enamel-etch technique

Apply phosphoric acid gel (e.g. N-Etch) onto the enamel and allow it to react for 15 – 30 seconds. Then rinse thoroughly with a vigorous stream of water for at least 5 seconds and dry with compressed air until the etched enamel surfaces appear chalky white.

#### (b.) Etch & rinse technique

Apply phosphoric acid gel (e.g. N-Etch) onto the prepared enamel first,

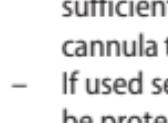
and then the dentin. The etchant should be left to react on the enamel for 15 – 30 seconds and dentin for 10 – 15 seconds. Then rinse thoroughly with a vigorous stream of water for at least 5 seconds and dry with compressed air until the etched enamel surfaces appear chalky white.

## 4. Handling VivaPen and bottle

### (a.) VivaPen

- Remove the VivaPen cap by turning it until noticeable resistance is felt and simultaneously pulling it off.

- Attach the VivaPen brush cannula. The cannula can be rotated up to approximately 270° to the desired position.

**Attention:** Rotating the cannula past the point of resistance will detach the cannula!

- For hygienic reasons, always use a VivaPen protective sleeve by sliding the Pen with the cannula into the protective sleeve until the cannula punctures through the sleeve.
- A new cannula and protective sleeve must be used for each patient.
- Activate the click mechanism 2 – 3 times until the cannula tip is sufficiently coated with Tetric N-Bond Universal. The wetting of the cannula tip is recognized by the yellow colour of Tetric N-Bond Universal.
- If used several times on the same patient, the moistened cannula must be protected from light.
- Remove the protective sleeve and cannula after using the VivaPen. Remove the cannula by turning it until noticeable resistance is felt and simultaneously pulling it off.
- Close the VivaPen with the original cap.

# Tetric® N-Bond Universal

**EN Instructions for Use**  
– Light-curing single-component dental adhesive

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## (b.) Bottle

- Dispense the desired amount of Tetric N-Bond Universal into a mixing well and apply it using a disposable applicator (e.g. Vivadent® Applicator Brush). Protect the adhesive from light (e.g. VivaPad®).
- Carefully close the bottle immediately after each use.
- A new disposable applicator must be used for each application.

## 5. Application of the adhesive

- Starting with the enamel, thoroughly coat the tooth surfaces to be treated with Tetric N-Bond Universal.
- The adhesive must be scrubbed into the tooth surface for at least 20 seconds. This time must not be shortened. Applying the adhesive on the tooth surface without scrubbing is inadequate.
- Disperse Tetric N-Bond Universal with oil- and moisture-free compressed air until a glossy, immobile film layer results. **Important:** Avoid pooling, since this can compromise the fitting accuracy of the permanent restoration.
- Tetric N-Bond Universal can either be light-cured for 10 s at a light intensity of 500 to 1,400 mW/cm<sup>2</sup> or for 5 s at a light intensity of 1,800 to 2,200 mW/cm<sup>2</sup>.
- For the remainder of the procedure, the Instructions for Use for the restorative and cementation materials must be observed.

Light intensity	Exposure time
500 – 1,400 mW/cm <sup>2</sup>	10 s
1,800 – 2,200 mW/cm <sup>2</sup>	5 s

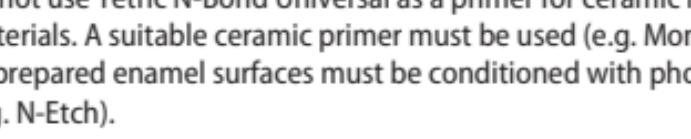
## 6. Intraoral repair of fractured composite restorations

### with light-curing materials

- Roughen the restorative surface to be repaired with a diamond finishing bur, clean with water spray, and dry with oil- and moisture-free compressed air.
- The subsequent working steps are the same as the application used for direct / indirect restorations with Tetric N-Bond Universal.

### Notes

- When using the VivaPen for the first time, more than 2 – 3 clicks are required to coat the cannula due to its design. This does not have any influence on the specified contents or the function of the VivaPen.
- Do not immerse the VivaPen in disinfecting solutions or autoclave it.
- Contamination on the VivaPen or bottle can be removed, e.g. by wiping with alcohol or other non-oxidizing disinfectants.
- For hygienic reasons, VivaPen protective sleeves must always be used.
- Cannulas other than those supplied with the VivaPen are not compatible with the VivaPen and must not be used.
- The fill-level indicator shows the remaining amount of material in the VivaPen. When the white indicator disappears from the see-through window, the VivaPen is empty.

- Do not use the bottle intraorally. For hygienic reasons, a suitable application aid should be used (e.g. VivaPad).
- Do not use Tetric N-Bond Universal as a primer for ceramic restorative materials. A suitable ceramic primer must be used (e.g. Monobond N).
- Unprepared enamel surfaces must be conditioned with phosphoric acid (e.g. N-Etch).

### Warning

- Tetric N-Bond Universal is an irritant. Avoid contact with the skin, mucous membrane and eyes.
- In case of skin contact, immediately wash with soap and copious amounts of water.
- If the material should accidentally come in contact with the eyes, immediately rinse with copious amounts of water and consult a physician / ophthalmologist.
- In rare cases, contact with skin may lead to sensitization towards the ingredients.
- Commercial medical gloves do not provide protection against the sensitizing effects of methacrylate.

## **Notes regarding phosphoric acid**

Phosphoric acid gel is corrosive. Avoid contact with the skin, mucous membrane and eyes (goggles are recommended for both the patients and practitioners). If the material should accidentally come in contact with the eyes, immediately rinse with copious amounts of water and consult a physician / ophthalmologist.

### **Storage**

- Expiration date: see imprint on packaging.
- Immediately close the VivaPen or the bottle with the original cap after each use.
- Storage temperature: 2 – 28 °C / 36 – 82 °F.

### **Keep material out of children's reach.**

### **For use in dentistry only.**

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

**The Material Safety Data Sheet is available online at**

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**Ivoclar Vivadent Marketing Ltd.**
**Ivoclar Vivadent AB**
**Ivoclar Vivadent Limited**
**Ivoclar Vivadent, Inc.**